


Getting Started

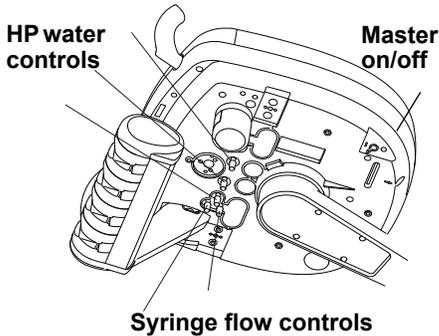

1/ Turn the unit on with the main on/off, toggle be sure to have all HPs in their holders or a beep error will sound and this message will appear briefly.

- * The HPs out of their holders are now locked out!
- * Place the HP back in their holders and turn off the unit, wait until all indicator lights are out, turn the unit back on.



MAIN

F




2/ Choose the user, press the F function button once.
 * The user, Dr number will be displayed as shown
 * Choose user # 0, 1, 2, or 3 pressing program buttons



Display of Dr number

3/ Program the preset chair positions for each user.
 * Use the manual chair controls to place the chair seat and back positions as desired
 * Press and hold the 1 program button until a beep sound is heard (10 sec.) to set the chair program
 * repeat to set the 2 and 3 chair programs as desired

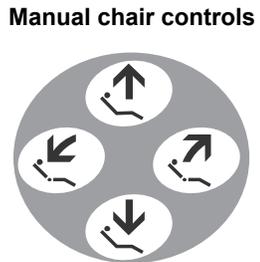


Note! touching any button will cancel auto chair movement for safety

4/ Pick up a HP and Choose water spray on or off and Choose Power optic on or off for each HP by pressing the buttons shown, indicator lights will go on.
 * Choose for each HP and the unit will remember, press the memory button and it will be memorized for next start up for that user.
 * The PO light can turn on holder pick up or foot control activation Press the F button 4 times, select - for foot control and + for holder




5/ Dental light (if chair mounted) button will turn the light on, hold for 2 sec to switch to enter no set composite mode.
 * press again to turn off or hold 2 sec to exit no set mode.

Note chair mounted lights have a manual switch located on the lower light swing arm which should be in the sensor position for normal use but can get accidentally switched either off or manual during cleaning, check this if the light does not come on.

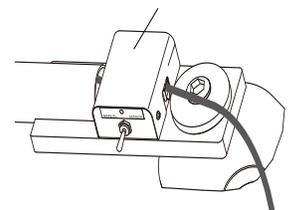

Override Light Switch

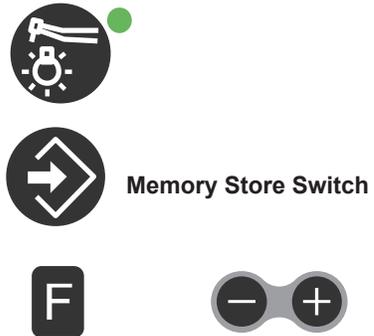



Cupfiller Switch



Bowl flush SW

6/ Cuspidor controls (if equiped) cup fill button will turn on for a preset time, bowl flush will run for a preset time.
 * press again to turn off either, or, hold bowl flush button down 3 sec to allow bowl flush to run indefinitely until pressed again.
 (times are programmable see manual)



Display for cup and bowl




7/ Timer use press the Timer button on the display to access the timer function. A beep sound will indicate time up.


- * Use the select button to choose a preset time as shown here.
- * Use the - & + buttons to select user chosen time.
- * Press timer button again to start timer, count down will display



Selection of time till beep

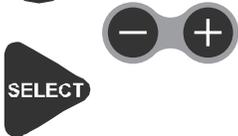

Variable Dial Knob

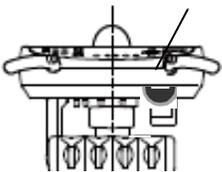

8/ Electric motor basic use
 ** see the full operation manual for many more details of micromotor use

- * Pick up the micromotor and speed will be displayed .
- * Use the "Select" button to select to Variable by the knob shown here below the display panel, the display will show the speed selected as you rotate the knob
- * Use the "Select" button to select one of the preset speed choices 1 2 or 3, use the plus or minus to select an RPM speed for that fixed M speed



Current set speed

Set to Fixed M2



Current set speed



- * Press the "function" button while motor is out of the holder and a menu to select Torque direction of auto rotate , auto time and intensity will display.



Current set speed


Auto-Reverse

Torque set



Function Button

- * Press the "function" button again to return to the motor speed and torque set display as shown here to right

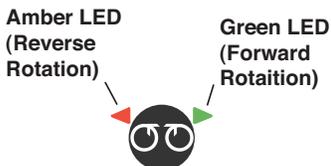

Forward-Reverse button selects direction of motor rotation

Safety Lock indicator will display when ever an handpiece is being run or the functions are being switched to prevent unwanted chair movement



Safety Lock Indicator